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Attenzione

## Sterile disposable sheaths for flexible and rigid cystoscope

Read these instructions for use carefully before each use and keep them easily accessible to interested users.

Please refer to the specific information on the intended use, indication and contraindications.

Read these instructions for use carefully. Improper use of the products may cause serious injury to the patient, user or third parties. Cover Srl rejects any responsibility for things or people in the event that other maneuvers are performed than those reported in this instruction sheet and in cases where the device was not used by qualified personnel.

### USER MANUAL

#### DEVICE DESCRIPTION

The **Cover-URO®** device is a sterile sheath to be used on the optical instrument called cystoscope. The sheath is equipped, in the distal part, with a lens to allow the vision with the optical instrument during the clinical activity by the urologist.

#### INTENDED USE AND CLINICAL BENEFITS

The sterile sheath for diagnostic optical instrumentation is used for the protection of optical instruments during hospital or outpatient endoscopic examinations. In particular, **Cover-URO®** is used in the urological field, on the instrument called cystoscope. The purpose of **Cover-URO®** is to prevent cross infections between patients and users due to the use of the instrument on multiple people during diagnostic examinations.

#### FEATURES

The cover sheath for cystoscopes **Cover-URO®** is available in various sizes suitable for use with the most popular models of cystoscopes on the market. The nominal diameters of the **Cover-URO®** sheath refer to intact cystoscopes and/or in any case corresponding to the specifications provided by the respective manufacturers.

The cover sheath for cystoscopes, **Cover-URO®** is a valid system to ensure simultaneously:

- The complete sterility of the cystoscope at every use
- The drastic reduction in recovery time between exams
- The reduction of procedure times

The system **Cover-URO®** is practical and easy to use: it fits easily on the cystoscope before use and, thanks to the reduced thickness and the great elasticity of the material used, it is easily tolerated by the patient even during the orientation phases. The high transparency of the lens and its conformation ensure a clear and distortion-free vision.

The device is **STERILE** and is **DISPOSABLE**.

#### INDICATIONS

The use of the **Cover-URO®** sheath is indicated for the execution of endoscopic urological examinations in inpatient and/or outpatient area through the use of cystoscopes and is reserved for adequately trained and qualified medical personnel to perform endoscopic examinations.

It can be used on adult patients only based on the size or article number chosen.

#### CONTRAINDICATIONS

- Do not use the **Cover-URO®** sheath with damaged cystoscopes.
- Using an unsuitable **Cover-URO®** sheath can damage the cystoscope.
- Do not introduce lubricants or detergents into/on the **Cover-URO®** sheath
- Do not force the introduction of the cystoscope into the **Cover-URO®** sheath

#### PRECAUTIONS

- In the case of modified and/or repaired equipment, please check the actual diameter of the equipment before using the **Cover-URO®** sheath.
- Before use, check the integrity of the primary packaging (envelope). Quality and sterility can only be guaranteed for sealed and undamaged packaging. If the primary packaging is open or damaged, discard the product and use a new one.

- Before use check expiry date on label and do not use after expiration date.
- Disposable device. Reuse can cause cross contamination for patient and operator.
- Do not re-sterilize. Re-sterilization may cause cross contamination for the patient and operator.
- Before use make sure you have a suitable sheath for the instrument to be covered.
- After covering the instrument with the sheath, visually verify that there are no visible cuts or holes.

#### INSTRUCTIONS FOR USE

##### Preparation:

Open the package and place the device on a serving trolley protected by a sterile drape

##### Positioning:

Hold the device through the protective tube and the proximal connector.

Slide **Cover-URO®** onto the instrument preferably in a horizontal position.

To facilitate the sliding of the instrument it is recommended to pre-lubricate with gel the operational part of cystoscopes

(NOT THE CONE - NOT THE DISTAL END).

Fix the connector to the cystoscope cone through a slight rotation.

Connect the camera and the light source.

Unwind the camera cover that covers the instrument body, the camera and a cable segment

Remove the protective tube and make sure that the green connector is securely attached to the cystoscope cone.

Remove the white cap from the Y-shaped connection piece and connect the pump tubing of the bag; open the flow to verify the correct passage of the solution.

Perform the cystoscopy.

##### Removal

Disconnect the device by slightly rotating the connector

Gently pull out **Cover-URO®** preferably from the distal lens.

#### REPORTING OF MAJOR ACCIDENTS

The user and/or the patient is recommended to report any serious incident that has occurred in relation to the device to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

#### CONSERVATION

If stored in the original packaging, the medical device is valid for 5 years.

The medical device can be stored and transported in a dry place, in its original packaging. Also protect from sunlight, heat and handle with care.

#### DISPOSAL METHODS

Disposable medical device. After use, throw the product into the appropriate hospital containers and according to the protocols of the health facility where the product is used. Do not disperse in the environment.

#### SYMBOLS USED ON THE LABELS OF THE PRODUCT

Do not reuse- Disposable

Medical Device

Expiry date

Do not use if the envelop is open or damaged

Keep away from light and heat sources

Manufacturer

Date of manufacture

Sterilized by EtO. Single layer sterile barrier system

Refer to the instructions for use

Keep dry

Product code

Lot number

Do not re-sterilize

Medical device according to the Regulations UE 2017/745

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